

Bedhead Call Button Release Note

(2026-01-12)


Device Model	DS-KHH6020X、DS-KHH6020X-H
Firmware Version	V2.3.11_build260112

New Features

1. Add support for Nurse on duty button and Code Blue alarm button to Bedhead call button. The connection method is Alarm input, with a short press to trigger and a long press of 3s to cancel. The Code Blue trigger depends on the Nurse on duty button already being triggered;
2. The alarm button wiring instructions for the bedhead call button must follow the one-to-one correspondence in the diagram. This is hardcoded in the new firmware:

Wiring diagram of alarm button

Alarm and RS-485 Port

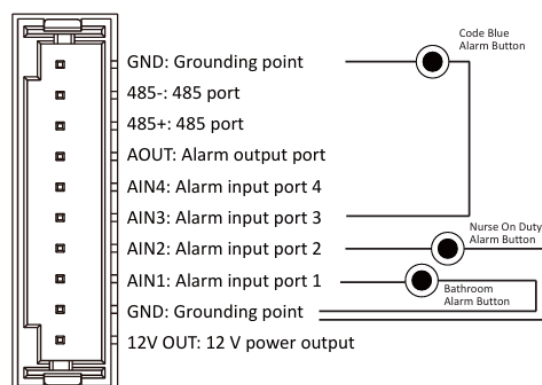

3. The Bedhead call button have added support for the automatic cancellation function of the Nurse on duty when the timeout period is 15 minutes.

Customer Impact and Recommended Action

This new firmware upgrade is to improve product performance, and will take effect automatically after upgrading from previous versions. We'd like to inform you the above changes. Also, we are sorry for any possible inconvenience of use-habit changes caused by this action.

For questions or concerns, please contact our local technical support team.

Note:

- Hikvision has all rights to alter, modify and cancel this notice.
- Hikvision doesn't give any guarantee for old models' stock.
- Hikvision is not liable for any typing or printing errors.
- For special model's change details, please contact our local technical support team.

Hikvision Digital Technology CO., Ltd.
No. 555 Qianmo Road, Binjiang District, Hangzhou 310052,
China
Tel: +86-571-8807-5998
FAX: +86-571-8993-5635
Email: support@hikvision.com